

Phonak ActiveVent™ Receiver

Candidacy checklist

Patient name:

Date:

DOB:

Background

ActiveVent Receiver features a switchable vent, which combines the benefits of an open fit and a closed fit in one receiver. Due to the complex nature of this technology, please consider the following:

ActiveVent Receiver requires:


- Hearing thresholds within the appropriate range (see diagram)
- Rechargeable Audéo Paradise hearing aids
- Additional cleaning and care beyond a standard receiver

Is ActiveVent Receiver right for your client?

Ask the following questions to determine if ActiveVent is the right solution for your client.

1. Does the client have hearing thresholds within the above range?
2. Would the client normally be fit with an open or a vented dome?
3. Does the client have sufficient dexterity to change wax filters?
4. Are the client's ears free of excess moisture and cerumen?
5. Is the client willing to disable audio notifications on their phone?
6. Does the client report problematic tinnitus?
7. Does the client report pain or discomfort from loud sounds?

Optimal fitting range for ActiveVent Receiver

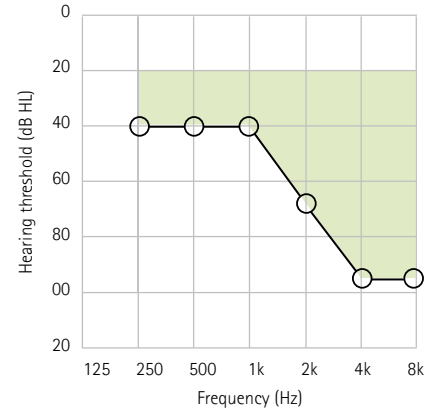

- | | |
|------------------------------|-----------------------------|
| <input type="checkbox"/> Yes | <input type="checkbox"/> No |
| <input type="checkbox"/> Yes | <input type="checkbox"/> No |
| <input type="checkbox"/> Yes | <input type="checkbox"/> No |
| <input type="checkbox"/> Yes | <input type="checkbox"/> No |
| <input type="checkbox"/> Yes | <input type="checkbox"/> No |
| <input type="checkbox"/> Yes | <input type="checkbox"/> No |
| <input type="checkbox"/> Yes | <input type="checkbox"/> No |

OK to proceed with ActiveVent Receiver trial and fitting?

- | | |
|------------------------------|-----------------------------|
| <input type="checkbox"/> Yes | <input type="checkbox"/> No |
|------------------------------|-----------------------------|

ActiveVent Receiver proficiency checklist

Before sending your client home with ActiveVent Receivers, they should be able to independently:

- Verify the comfort of the Universal SlimTip AV or Titanium SlimTip AV
- Remove and insert ActiveVent Receiver into/out of the ear
- Wipe away any cerumen or debris on the SlimTip AV
- Change the HF3 wax filter
- Confirm the loudness of the vent closing as acceptable
- (Optional) Recognize when a cleaning reminder is received in the myPhonak app

If the client is unable or unwilling to do any of the following, we recommend to continue the fitting with a standard SDS 4.0 receiver.