

Phonak Fast Facts



Phonak Bolero™ B-PR

What is it?

Phonak Bolero B-PR is a fully featured BTE hearing aid with a telecoil, volume control and a built-in rechargeable battery, eliminating the need to replace batteries. Bolero B-PR offers 24 hours* of battery life on a single charge by using a lithium-ion (Li-ion) battery. Additionally, the new housing design offers the benefits of more durability with an IP rating of 68 (water resistant and dust tight).

Why use it?

With rechargeable batteries, clients benefit by having hassle-free hearing without having to change batteries or carry extra batteries with them. Compared to other rechargeable technologies, Li-ion batteries have the additional benefit of reduced deterioration and degradation of the battery over time without affecting hearing aid performance. Furthermore removing the battery door has improved the durability of the hearing aid. You also benefit by having more counselling time which can be spent on topics not relating to battery changing and maintenance.



How to use it?

Bolero B-PR is easy to use. Simply remove the hearing aids from the charger and press the push button for 3 seconds until the light indicator briefly flashes green. The hearing aids are now switched on and ready for use. At the end of the day, simply place the hearing aids into the charger. The hearing aids will click into the slots and automatically switch off. The green light indicator flashes slowly to indicate charging. Once the light stops flashing and remains solid the hearing aids are fully charged and ready to for use. Charging time for a completely depleted hearing aid is 3 hours.

For more information on the Phonak Bolero B-PR and its charging options, go to www.phonakpro.com/bolero-b or contact your Phonak representative.

*Expected results when fully charged, and up to 80 minutes wireless streaming time. Please refer to www.phonakpro.com/evidence