

Phonak Fast Facts



iCube II

What is it?

iCube II is a cable-free fitting device that wirelessly transfers fitting data from the PC to the client's hearing aids. It can be used with any wireless Phonak hearing aid (Venture, Quest, Spice+ and Spice).

Why use it?

The fitting process with iCube II is fast, smooth and cable-free. With improved reliability and stability, clients can enjoy the freedom to move or to walk around without being tangled up by cables.

The dedicated USB adapter makes it easy to connect iCube II to the fitting PC. It is as simple as installing a wireless mouse. With the new neckloop that connects in the middle, it is now much easier to put iCube II on. The significantly optimized battery consumption allows for up to twelve hours of continuous fitting without the need to recharge.

How is it used?

iCube II is truly Plug&Play with the pre-paired USB adapter. To get started, the adapter only needs to be plugged into a free USB port on the fitting PC. The connection will be established automatically when iCube II is selected as fitting device in Phonak Target 4.0 (or newer).

It can also be used with a USB cable or connected wirelessly via Bluetooth to any other PC or tablet.



For more detailed information visit:
www.phonakpro.com/icube