

# Using the Roger DigiMaster X

## Reference Sheet



School: \_\_\_\_\_

Teacher: \_\_\_\_\_

Roger contact: \_\_\_\_\_

Phone: \_\_\_\_\_

Date: \_\_\_\_\_

Serial number: \_\_\_\_\_



## Connecting

### Initial set-up








1. Connect the Roger DigiMaster X to a power source.
2. Connect the Roger DigiMaster X to the room amplification system.
3. Hold Roger inspiro less than 10 cm (4 inches) from the Roger DigiMaster X and press **Connect**.



### Daily use

1. Press the power button  on the Roger DigiMaster X for 2 seconds to power on. Turn on Roger inspiro. Speak into the Roger inspiro microphone.
2. Press the power button  on the Roger DigiMaster X for 2 seconds to power off when finished.

## Roger DigiMaster X indicator light

-  **Solid red** – Roger DigiMaster X is off
-  **Solid blue** – Standard operation mode
-  **Blinking blue** – Roger microphone is off or out of range
-  **Solid green** – Microphone is muted
-  **Green to red** – Volume is adjusting +/-
-  **Blinking yellow** – Software update
-  **Blinking red** – Call your Roger contact.