
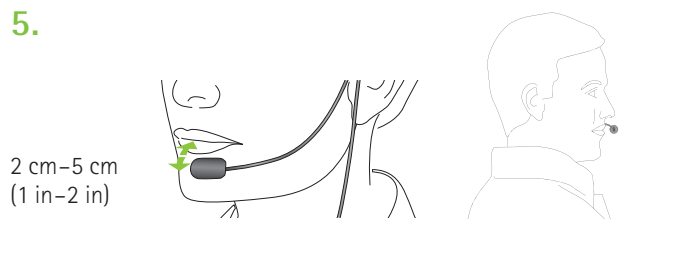
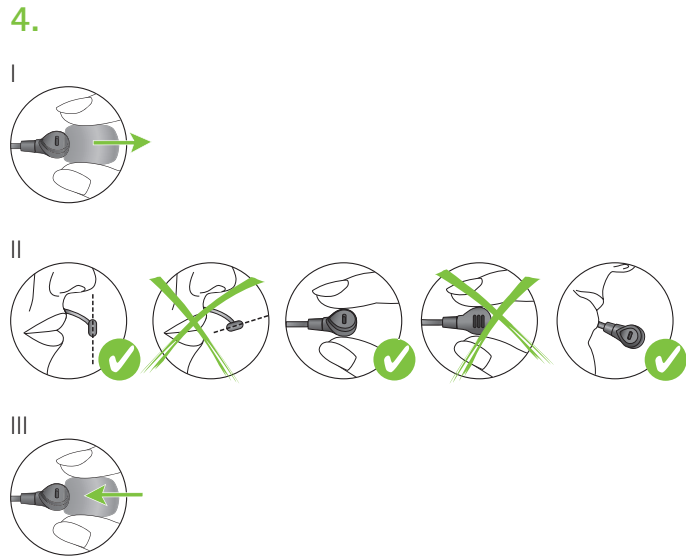
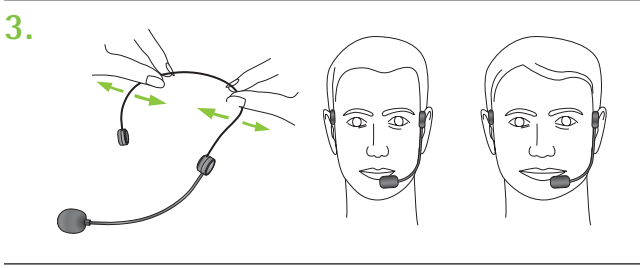
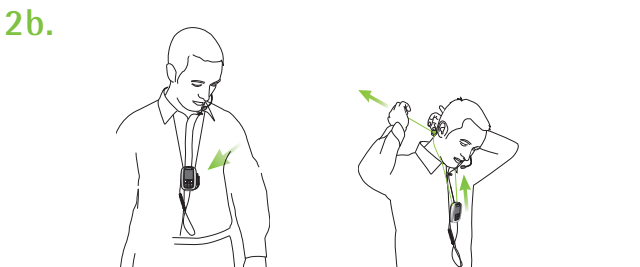



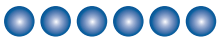



# Getting Started

## Dynamic SoundField

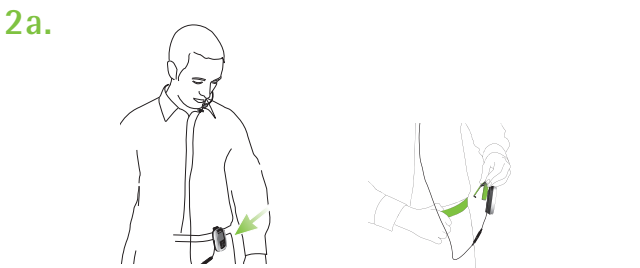

8. Ready, start talking.

### DigiMaster 5000 LED status indicators

-  **Constant blue light:**  
Standard operating mode
-  **Blue blinking light:**  
Power-saving stand-by mode
-  **Yellow blinking light:**  
inspiro not paired
-  **Constant green light:**  
inspiro mic muted
-  **Constant red light:**  
DigiMaster 5000 needs re-starting

[www.Phonak.com](http://www.Phonak.com)  
[www.DynamicSoundField.com](http://www.DynamicSoundField.com)