

Phonak Fast Facts



TV Connector

What is it?

The TV Connector is the easy plug and play streamer that directly connects Phonak Audéo™ B-Direct hearing aids to any TV or media player without the need of a body worn streamer. With AirStream™ Technology, the new Phonak proprietary 2.4 GHz streaming protocol, TV Connector offers a direct audio streaming from a broad range of TV and audio equipment in excellent stereo sound quality to Audéo B-Direct.

Why use it?

Benefits to your clients

- No body worn streamer required
- Transforms Audéo B-Direct hearing aids into wireless stereo headsets
- Easy plug and play setup
- Provides high quality speech and music from any audio source.
- Environmental balance can be used to adjust the volume of the streamed signal or surrounding environmental sounds
- Unique advantage of connecting multiple listeners simultaneously to the same audio signal.
- Up to 80 hours of TV listening with one hearing aid battery

How to use it?

The TV Connector is plug and play right out of the box to any TV with optical or 3.5mm interfaces. The hearing aids are automatically paired to the TV Connector. If pairing is not automatic, a one-time push on the TV Connector pairing button will connect the hearing aids within 1m (3ft). To connect multiple listeners, press the pairing button on the TV connector and clients within 1m (3ft) of the TV connector will be automatically paired. Turn the TV on and when the client is within a range (15m/49ft) from the TV Connector, streaming begins automatically.



For more information on the TV Connector, go to www.phonakpro.com/tvconnector or contact your Phonak representative.